


SIZE GUIDE

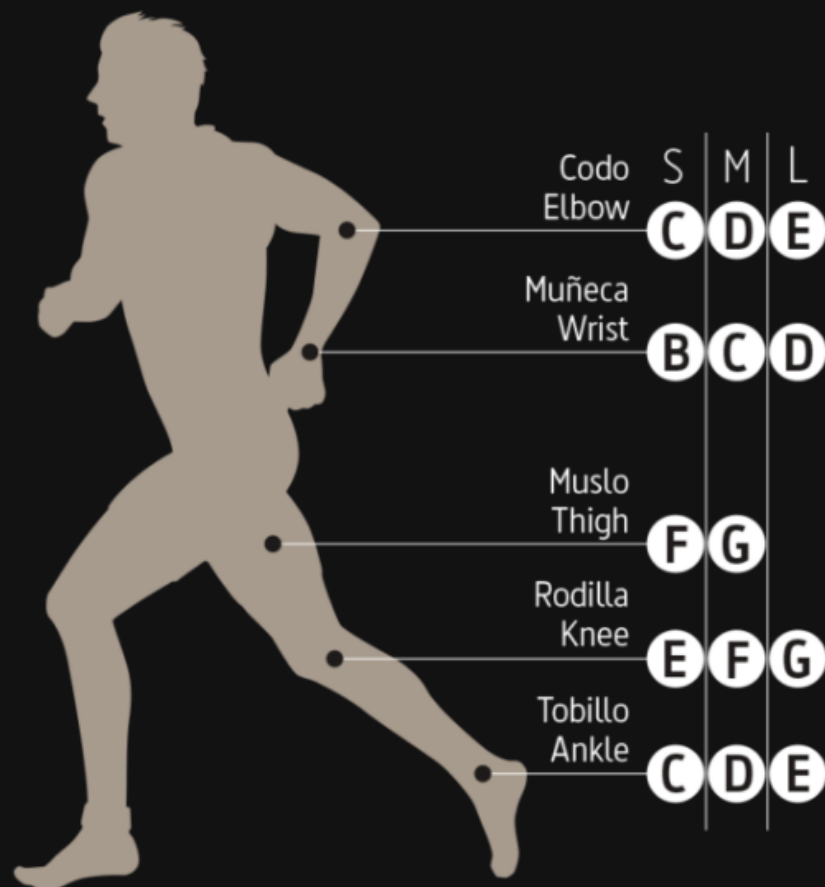

Size	Black	Blue	Orange	Pink
B	60501	60507	60513	60519
C	60502	60508	60514	60520
D	60503	60509	60515	60521
E	60504	60510	60516	60522
F	60505	60511	60517	60523
G	60506	60512	60518	60524

Instructions for use Composition

1. Measure the widest part where the bandage must be placed with the help of the tension guide (if available) and select the proper size of the bandage according to the desired compression.
2. Cut twice the required length for the desired area to be bandaged.
3. Put the bandage on the desired part of the body and double it back over limb to over-layer it. Allow an extra 3 cm to the upper layer to overlap the lower on.