How to use your new Rehband Wrist Support - 7080*, 7710*, 7793*, 7910*

*different colors, sizes, thicknesses, sides

7080*

7710*

7793*

7910*

EN

Function

Gives warmth and compression to the wrist. Stabilizes and increases muscle co-ordination in the area.

Indication

Strains, wear-and-tear injuries, soft tissue overload of the hand or forearm.

Application

- 1) Place the support on the wrist.
- 2) Close all velcro fasteners.
- 3) Make sure the support sits firmly without pressing.

Important Information

- 1) If you have any questions regarding application, please contact your local dealer or your health-care practitioner.
- 2) The product is not flameproof; avoid temperatures above 120°C / 248°F.
- 3) The function of the product must not be changed or compromised; the product should only be used for its original, intended purpose.
- 4) The SBR/neoprene product-use period should not exceed 3-4 hours.
- 5) If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.
- 6) Product 7710* is containing splints.

Washing Instructions

- 1) Close all velcro fasteners.
- 2) Remove detachable splints 7710*
- 3) Wash at 40 °C with similar colors.
- 4) Mesh laundry bag is recommended.
- 5) Stretch product while wet and let dry.
- 6) Please follow care instructions on the products care label.

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