
How to use your new Rehband Back Support - 7082*, 7730*, 7930*

*different colors, sizes, thicknesses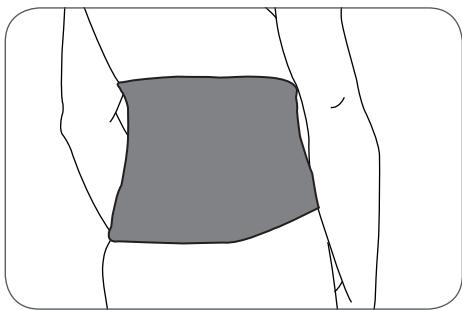

EN

Function

Gives warmth and increases the blood flow. Provides pain relief. Gives compression and increases muscle coordination in the area. Stabilizes the back.

Indications

Lumbago, back strains, wear and tear injuries, soft tissue overload. Severe backache, chronic pain. Extra stability for sporting exertion/heavy work.

Application

- 1) Place protection over the back.
 - 2) Take the lower edge of the left end of the protection.
 - 3) Take the right end and lay this over the left part in such a way that the velcro fasteners meet. Pull tight enough for comfort.
- Important Information**
- 1) If you have any questions regarding application, please contact your local dealer or your health-care practitioner.
 - 2) The product is not flameproof; avoid temperatures above 120°C/ 248°F.
 - 3) The function of the product must not be changed or compromised; the product should only be used for its original, intended purpose.
 - 4) The SBR/neoprene product-use period should not exceed 3-4 hours.
 - 5) If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.

Washing Instructions

- 1) Close all velcro fasteners.
- 2) Wash in 40 °C with similar colours.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry.
- 5) Please follow care instructions on the products care label.

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