# How to use your new Rehband Shoulder Support - 7731\*

\*different colors, sizes, thicknesses, sides

EN

## Function

Conveys heat to the shoulder, chest and neck muscles and stimulates blood circulation in the joints and joint region. Alleviates pain and reduces stiffness. Optimises muscle control in the shoulder area, stabilizing the shoulder joint activity.

## Indication

General shoulder joint instability due to soft tissue strains/ruptures, collar bone displacements. The support can be used during active shoulder rehabilitation.

## Application

- Follow the instructions on the pictures.
- Place the support on your shoulder.
- Retract your shoulders and close the chest strap.
- Pull and attach the V-shaped closure.
- Attach the arm anti-rotation strap on the adjustable arm closure. Adjust the strap if needed.

## Important Information

- Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- If you have questions regarding application, please contact your local dealer or the prescriber.
- Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120°C/ 248°F.
- Do not continue using a product that is no longer functional, or that is worn or damaged.

## Washing Instructions

- Wash in warm water at 40 °C (104°F) with similar colours.
- Use laundry bag or net.
- Stretch product while wet and allow to air dry.
- Clean the product regularly according to care instructions on the product care label.
- Fasten all velcro closures.

## Size selection

- Do the measurements as instructed on the packaging.
- Determine the size of the support (see size chart).

## Conformity declaration

This product meets the requirements of European Directive 93/42/EEC for medical devices.

## Disposal

Dispose the product in accordance with national regulations.

## Material

Foam: SBR/Neoprene or Neoprene (see on textile label)

Cover: polyamide (see on textile label)

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