How to use your new Rehband Back Support - 7732*

*different colors, sizes, thicknesses, sides

EN

Function

Gives support, compression and warmth to back. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides pain relief and stabilization. Suitable for strength sport activities and lifting heavy weights.

Indication

Over-exertion and soft tissue overload in the lower back area. Chronic and severe back pain, inflammation, wear and tear injuries.

Application

- 1) Place the support on the lower back.
- 2) Hold the left end from the lower edge of the support. Take the right end and place it over the left part. Tighten until you feel the desired pressure from the support.
- 3) Pressure on the back can be varied by adjusting two additional flexible straps, by pulling them either a little upwards or downwards before fastening on the stomach.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer or the prescriber.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C / 248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Remove detachable splints.

Size selection

- 1) Use the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Conformity declaration

This product meets the requirements of European Directive 93/42/EEC for medical devices.

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene (see textile label)
Cover: polyamide and polyamide-elastane (see textile label)
Plastic splints: polypropylene
Metal splints: steel

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