
How to use your new Rehband Elbow Support -102*, 7081*, 7720*, 7921*

*different colors, sizes, thicknesses, sides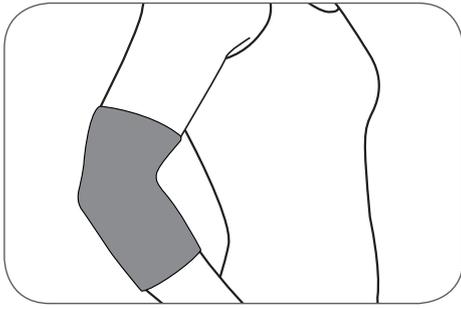

EN

Function

Gives warmth and increases the blood flow. Provides pain relief. Gives compression around the elbow joint. Increases muscle co-ordination in the area (proprioception).

Indication

Inflammatory problems, haematomas, soft tissue overload, muscle tears.

Application

- 1) Place the support on the elbow.
- 2) The curve of the lateral seam must be on the same level with the elbow joint.

Important Information

- 1) If you have any questions regarding application, please contact your local dealer or your health-care practitioners.
- 2) The product is not flameproof; avoid temperatures above 120°C/ 248°F.
- 3) The function of the product must not be changed or compromised; the product should only be used for its original, intended purpose.
- 4) The SBR/neoprene product-use period should not exceed 3-4 hours.
- 5) If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.

Washing Instructions

- 1) Mesh laundry bag is recommended.
- 2) Wash at 40 °C with similar colors.
- 3) Stretch product while wet and let dry.
- 4) Please follow care instructions on the products care label.

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