**NIPPON SIGMAX CO., LTD.**  
33rd Floor Shinjuku Oak Tower  
6-6-1 Nishi-shinjuku, Shinjuku-ku,  
Tokyo 163-6033 JAPAN  
Phone: +81 3 5326 3270

## ZAMST HA-1 INSTRUCTIONS FOR USE

**Please read through the following instructions completely before using the product.**

\*This product is for arch support; however, product fit varies by individual.

\*This product does not prevent or heal foot injuries.

### How to apply:

The socks are designed specially for the right and left feet. Make sure to wear the correct sock on each foot.

### Precautions:

- To prevent adverse symptoms, do not use the socks in the following cases:
  - If your skin is sensitive or prone to allergies.
  - If you have abnormal conditions such as cuts, fractures, numbness, swelling or rash in the area where product is used.
- Follow the instructions below and use the product correctly to avoid poor blood circulation or injuries.
  - Use the correct size. (Refer to the size chart on the box.)
  - Do not use the product while sleeping.
  - Do not use the product for extended periods.
- If you experience the following adverse effects, discontinue use immediately and consult with a physician.
  - If you experience discomfort while wearing the product.
  - If you experience cuts, fractures, pain, numbness, swelling or a rash in the area where this product is used.

### Caring for the product:

- Use a laundry bag when you clean the product in a washing machine.
- The product's dye may bleed. Do not wash with whites or bright colors.
- Do not place in dryer.

### Composition:

Cotton, acrylic, polyurethane, nylon and polyester

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